



Memo No. 2018-003
Date: 5-Jan-2018
Memo To: Clients
Re: Revised Tests

CA 15-3 (Abbott Architect CMIA)

The test listing has been revised to include:
Specimen Type: Serum
Stability: 1 day refrigerated, longer frozen.
Turnaround Time: 3 Days

Hemoglobin A1c (HbA1c)

Specimen handling instructions have been revised for both methods (TINIA and Capillary Electrophoresis) for improved specimen quality.
Submit original blood collection tube, do not transfer to transport tube (e.g. 4 mL Whole Blood, EDTA), do not aliquot. Submit specimen immediately - must be received at ICL Mon-Thu within 4 days of collection.

Pregnenolone

Specimen handling and reference values have been revised by the performing lab effective immediately:
Avoid gel separator tubes. Place sample on ice after collection and freeze separated serum as soon as possible. Store and send frozen.

Stability: Refrigerated 72h; Frozen 14 Days

Reference Range(s):

Adult 22-237 ng/dL

Pediatric

1-59 Days 68-1303 ng/dL

60 Days-1 Year ≤219 ng/dL

2-6 Years ≤140 ng/dL

7-9 Years ≤156 ng/dL

10-12 Years 15-220 ng/dL

13-17 Years 12-196 ng/dL

If you have further questions please contact Client Care at (416) 422-3000 Ext. 300 or info@ICLabs.ca

Shashank Tilak. Laboratory Director
(416) 422-3000 Ext. 221
ShashTilak@ICLabs.ca

Want to receive updates by e-mail? Please contact Client Care info@ICLabs.ca

ICL is the only Not-For-Profit dedicated broker of lab diagnostic tests in Canada

57 Gervais Drive, North York, ON M3C 1Z2

(416) 422-3000 Ext. 300 info@ICLabs.ca